

WOUND CARE DryMax[®]

EASY


DryMax Easy is a sterile conformable, superabsorbent dressing for use on exuding wounds. DryMax Easy absorbs and retains exudate in the dressing. The exudate is absorbed vertically by the dressing and retained in the core forming a gel, thus minimizing the risk of maceration and maintaining a moist wound environment.

DryMax Easy


The soft tack function helps the dressing to adhere to the skin and will work as an extra hand during the dressing application. The dressing can be applied with either side facing the wound. This feature also makes it possible to stack dressings on top of each other if extra absorption capacity is required. The seamless cover simplifies positioning over the wound.

The dressing remains dry and comfortable for the patient. Simultaneously it contributes to a moist wound environment. The hydrophilic contact layer transports fluid effectively away from the wound and distributes it evenly into the ABSORBEST core.


The ultra-thin and highly conformable features allows the dressing to be used on all parts of the body. DryMax Easy is designed to work well together with compression therapy.

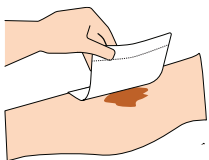

The dressing must be placed close to the wound, or placed in close combination with a suitable primary dressing.

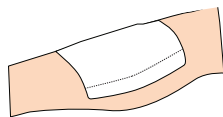

Secure the dressing with suitable bandage, film or tape.

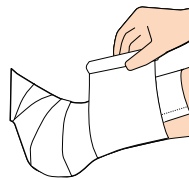

DryMax Easy can be used in conjunction with compression therapy.

Item	REF	Pcs/box	Product size (cm)	Absorption capacity*
DryMax Easy 10x10 cm	F60033/10	10	10x10	170 ml
DryMax Easy 10x20 cm	F60035/10	10	10x20	500 ml
DryMax Easy 20x20 cm	F60036/10	10	20x20	1000 ml
DryMax Easy 20x30 cm	F60037/10	10	20x30	1550 ml

*) According to international standard EN 13726



- Cost effective
- User friendly
- Increased patient comfort
- Improved wound healing



With Application aid.



ABSORBEST

Manufactured by Absorbest AB Sweden
www.absorbest.se | info@absorbest.se