

DryMax Soft

DryMax Soft is a sterile, conformable, superabsorbent dressing for exuding wounds. Its properties provide an easy-to-use and reliable solution for both patient and clinician. DryMax Soft absorbs and retains exudate in the dressing. The exudate is absorbed vertically by the dressing and retained in the core, forming a gel, thus minimizing the risk of maceration and maintaining a moist and balanced wound environment.

The seamless cover simplifies positioning over the wound. The design allows the dressing to be applied with either side facing the wound. This feature also makes it possible to stack dressings on each other if extra absorption capacity is required.

The dressing remains dry and comfortable for the patient. Simultaneously, it contributes to a moist wound environment. The hydrophilic contact layer transports fluid effectively away from the wound and distributes it evenly into the dressing's core constructed by our DryMax Technology.

The ultra-thin and highly conformable features allow the dressing to be used on all body parts. DryMax Soft is designed to work very well together with compression therapy.

Application Guide



The dressing must be placed close to the wound, or placed in close combination with a suitable dressing.



Secure the dressing with suitable bandage, film or tape.



DryMax Soft can be used in conjunction with compression therapy.

| Item | Product size cm | REF | Pcs/box | Absorption- capacity* |
|-------------|--------------------|-----------|---------|--------------------------|
| DryMax Soft | 7.5x7.5 | F60011/10 | 10 | 60 ml |
| DryMax Soft | 10x10 | F60013/10 | 10 | 130 ml |
| DryMax Soft | 10x15 | F60012/10 | 10 | 230 ml |
| DryMax Soft | 12,5x12,5 | F60123/10 | 10 | 220 ml |
| DryMax Soft | 10x20 | F60015/10 | 10 | 330 ml |
| DryMax Soft | 15x20 | F60014/10 | 10 | 400 ml |
| DryMax Soft | 20x20 | F60016/10 | 10 | 600 ml |
| DryMax Soft | 20x25 | F60018/10 | 10 | 780 ml |
| DryMax Soft | 20x30 | F60017/10 | 10 | 960 ml |
| DryMax Soft | 37x56 | F60019/10 | 10 | 3000 ml |

The average free swell absorption capacity* of DryMax Soft is 1.8 ml per cm² of the product core. *According to international standard EN 13726



- Cost effective
- User friendly
- Increased patient comfort
- Improved wound healing*



DryMax® technology



^{*}Compared to conventional wound dressings