

## Absorbest Relevo

Absorbest Relevo is a sterile, conformable, superabsorbent dressing for exuding wounds. Absorbest Relevo absorbs and retains exudate in the dressing. The exudate is absorbed vertically by the dressing and retained in the core forming a gel, thus minimizing the risk of maceration and maintaining a moist and balanced wound environment.

The design allows the dressing to be applied with either side facing the wound. This feature also makes it possible to stack dressings on each other if extra absorption capacity is required.


The dressing remains dry and comfortable for the patient. Simultaneously it contributes to a moist wound environment. The hydrophilic contact layer transports fluid effectively away from the wound and distributes it evenly into the dressing's core constructed by DryMax Technology.

The ultra-thin and highly conformable features allows the dressing to be used on all body parts. Absorbest Relevo is designed to work well together with compression therapy.


### Application Guide



The dressing must be placed close to the wound, or placed in close combination with a suitable dressing.

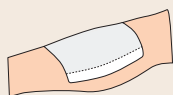

Secure the dressing with suitable bandage, film or tape.

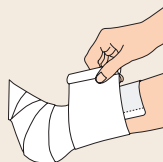

Absorbest Relevo can be used in conjunction with compression therapy.

Item	Product size (cm)	REF	Pcs/box	Absorptioncapacity*
Absorbest Relevo	10x10	F50011/10	10	50 ml
Absorbest Relevo	10x10	F50011/50	50	50 ml
Absorbest Relevo	10x20	F50021/10	10	130 ml
Absorbest Relevo	10x20	F50021/50	50	130 ml
Absorbest Relevo	20x20	F50041/10	10	250 ml
Absorbest Relevo	20x20	F50041/50	50	250 ml

\*According to international standard EN 13726

\*\*Compared to conventional wound dressings



- Cost effective
- User friendly
- Increased patient comfort
- Improved wound healing\*\*



**DryMax® technology**